

## Condition Guidelines for Programming METIman

This card is intended to help you select Müse conditions to achieve desired vital signs within each programmed state. All four conditions should be programmed into each state in the order presented below.

- Respiratory: Desaturation
- Cardiovascular: Blood Pressure
- Cardiovascular Heart Rate
- Respiratory: Respiratory Rate

The Müse software is physiologically driven. When using multiple conditions (e.g., Desaturation + Hypertension + Tachycardia + Tachypnea) the Physiological Baroreceptors cause compensatory changes within parameters. To achieve the desired vital sign, select one condition level, above (greater) or below (less), to achieve the desired physiological effect.

### Respiratory: Desaturation

Desaturation	SpO <sub>2</sub> Value
Reset	98%
High 90s	96-97%
Mid 90s	94-96%
Low 90s	91-93%
High 80s	87-90%
Mid 80s	84-86%
Low 80s	80-83%
High 70s	77-80%
Mid 70s	74-77%
Low 70s	69-71%
Less than 70	<69%

**Cardiovascular: Blood Pressure**

<b>Hypertension</b>		<b>Hypotension</b>	
Reset	110s/70s	Reset	110s/70s
Increased	120s/80s	Decreased	100s/70s
Pre-Borderline	130s/80s	Pre-Borderline	100s/60s
Borderline	140s/90s	Borderline	90s/50s
Mild	150s/90s	Mild	80s/40s
Moderate	160s/100s	Moderate	70s/40s
Severe	170s/100s	Severe	60s/30s
Profound	190s/110s	Profound	50s/30s
Extreme	220s/120s	Extreme	40s/30s

**Cardiovascular: Heart Rate**

<b>Tachycardia</b>		<b>Bradycardia</b>	
Reset	70s	Reset	70s
Increased	High 70s	Decreased	Mid 60s
Elevated	80s	Pre-Borderline	Low 60s
Pre-Borderline	90s	Borderline	Mid 50s
Borderline	100s	Intermediate	Low 50s
Intermediate	110s	Mild	High 40s
Mild	120s	Moderate	Mid 40s
Moderate	130s	Severe	Low 40s
Severe	140s	Extreme	Mid 30s
Supra	150s	Acute	Low 30s
Profound	160s		
Extreme	170s		
Acute	High 170s		

**Respiratory: Respiratory Rate**

Tachypnea		Bradypnea	
Reset	11	Reset	11
Increased	15	Increased	10
Elevated	18	Intermediate	9
Borderline	20	Mild	7
Intermediate	22	Moderate	6
Mild	25	Severe	5
Moderate	28	Profound	3
Severe	31	Extreme	2
Profound	33		
Extreme	36		