


Installing System Updates in Müse

System updates are available periodically. Once acquired, the update must be installed on the Müse software. The update file is an *.msu file that is downloaded previously from the Internet.

To install a system update in Müse:

1. Log into Müse.

The Home page appears.

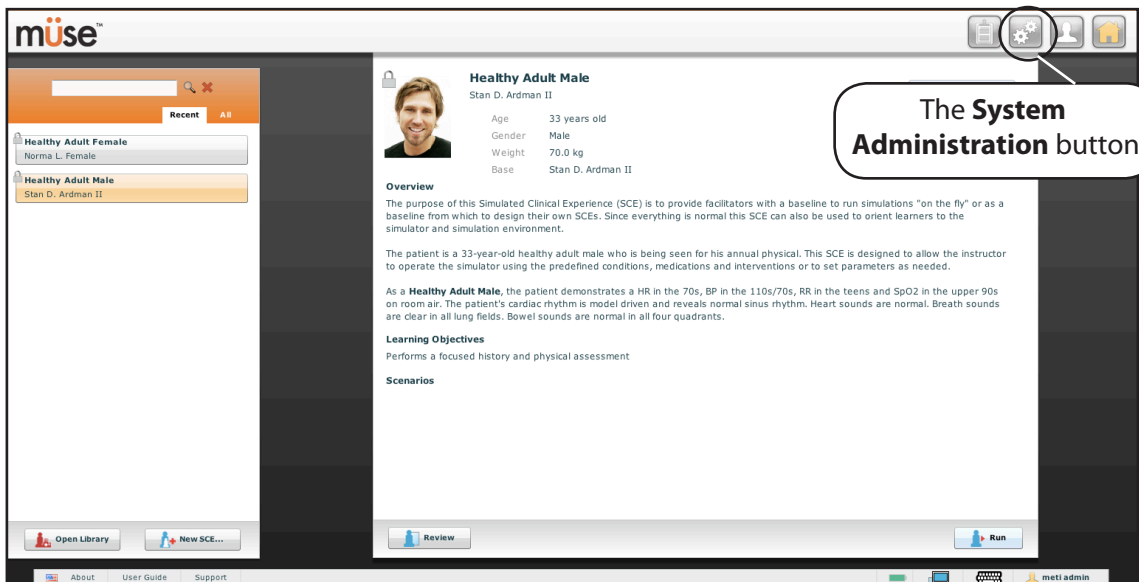

The Home Page

2. From the Home page, in the top, right-hand corner, click the **System Administration** button.



The System Administration Button

The System Administration screen appears.


3. Click the **System Settings** button.

Click **System Settings**



Clicking the System Settings Button

The System Settings panel appears.

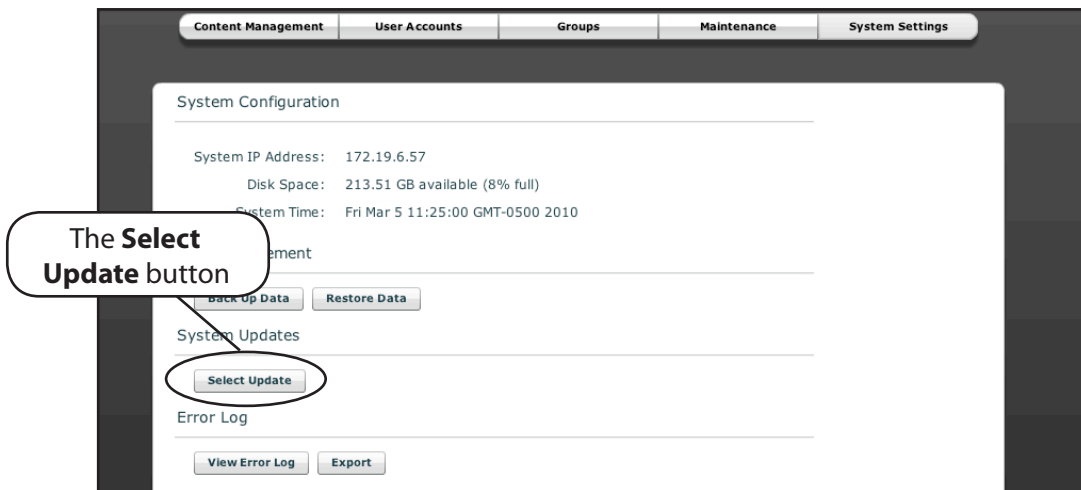


The System Settings Panel

4. From the System Settings panel, click **Select Update**.

A select file dialog box appears.

5. Select the (*.msu) update file.

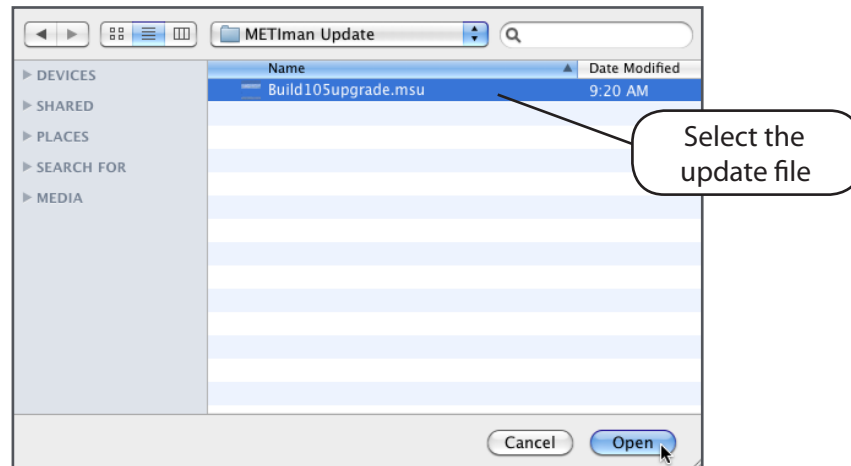


Selecting the Update

The software performs the update. This process may take a few moments.

Once the update has finished, a message appears telling you to reboot.

- Restart METIman.
- Restart the Instructor Workstation.
- From Instructor Workstation, launch Müse.
- From the Müse Start screen, empty the cache. In Safari, click the **Safari** menu, and select **Empty Cache**.

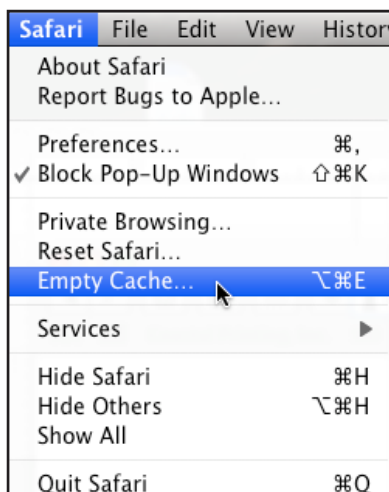

Selecting Empty Cache

For help with any other browser on how to empty the cache, please consult your browser's help menu.

The update has been performed, and the software is ready to use.