

METIVision[®] Version 1.0.16 Update

This install guide provides instructions on how to install the METIVision Version 1.0.16 update automatically from a METIVision that is connected to the Internet and directions for users who wish to download the update from the METI website.

METIVision Version 1.0.16 has these new features:


- Müse software compatibility
- Network backup support capability
- Enhanced hardware status notification
- Secured web access
- Streamlined management of simulators on a Central Server
- Manual ability to enable and disable automatic clock synchronization
- Support for the Axis 214 and Vivotek PZ71X1 cameras
- Support for the Axis Q7401 video encoder

IMPORTANT: You must have installed the latest version of Müse – iStan Müse 1.1 (55.1) or METIman Müse 1.1 (77) – in order to utilize METIVision with the Müse software. All installations of Müse should be completed before installing the METIVision version 16 update.

Online Update Procedure


This update is for METIVision users whose console is connected to the Internet and who have version 15 installed.

1. Perform steps 1 to 3 on the METIVision Setup map.
2. Log into the METIVision Main Setup screen by entering the address `http://1.1.1.1/console/` into the address bar of your web browser.
3. Locate the Update heading on the METIVision Main Setup screen.
4. Click the **Get and Install Online Updates** button.

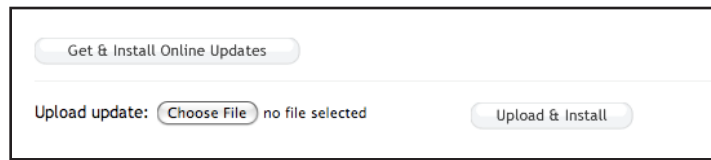

Clicking the Get and Install Online Updates Button


The text “Update in progress...” shows below, indicating the update is active. The update process takes approximately five to 10 minutes depending on the Internet connection speed.

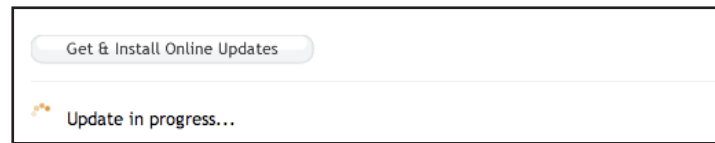

Updating METIVision


IMPORTANT: Do NOT reboot your METIVision or refresh the web browser while the update is installing.

Once the update is finished, the message “Last Update was Successful” is displayed.

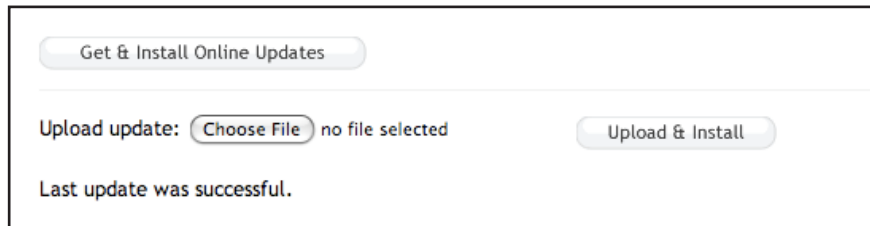

Completing the METIVision Update

5. Exit the METIVision Main Setup screen by closing the web browser.
6. Remove the Ethernet Cable from the Camera port.
IMPORTANT: If the computer being used for this update is the same as the one being used for the METIVision, the Network settings must be restored. Contact your Network Administrator if you need assistance. Once restored, proceed to Step 7. If using an alternate computer to access METIVision, proceed to Step 7.
7. From the computer used to access METIVision, using the METIVision IP address, access the METIVision Login screen.
The METIVision Login screen appears.
8. To confirm the update is complete, click the **About** box in the lower, right-hand corner of the METIVision Login screen.
METIVision Version 1.0.16 is displayed.

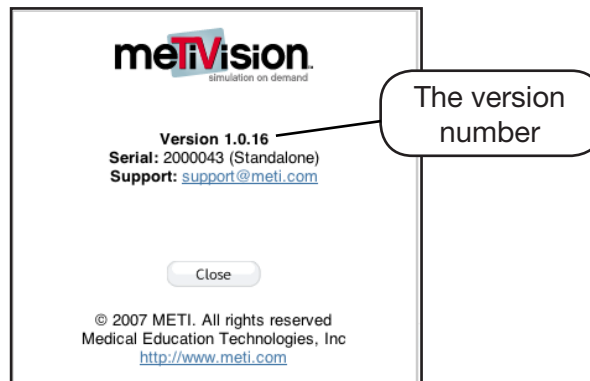

The About Box

Downloadable Update Procedure

This update is for METIVision users running version 15 or higher. The computer used to download the update must have access to the Internet.


1. From a computer being used to update METIVision, launch the web browser and access the link http://www.meti.com/mymeti/downloads_main.html.
2. If using a Macintosh Operating System, holding the **Control** key, click the **METIVision UPDATE v1_0_16.gpg** link. If using a Windows Operating System, right-click the **DOWNLOAD METIVision UPDATE v1_0_16.gpg** link.

A pop-up menu appears.

3. Depending on the browser used, click either **Save Link As**, or **Save Linked File As** or **Save Target As**.


A dialog box appears asking where you would like to save the file.

4. Save the update file in an easily accessible location.
5. Close the web browser.
6. Perform steps 1 to 3 on the METIVision Setup map.
7. Log into the METIVision Main Setup screen by entering the address <http://1.1.1.1/console/> into the address bar of your web browser.
8. Locate the Update heading on the METIVision Main Setup screen.
9. Click the **Choose File** button.

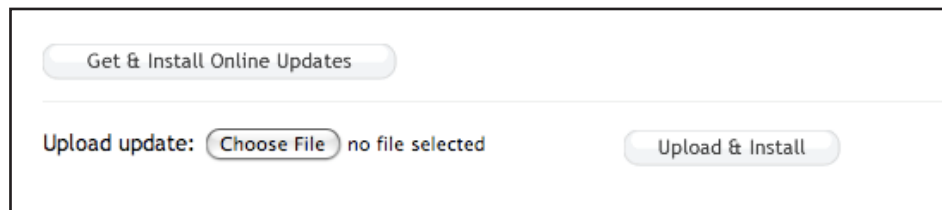

Clicking the Choose File Button


10. Select the update file **METIVision_Install_Upgrade_v1_0_16.gpg** and select **Choose**.

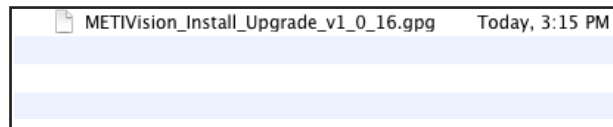

The METIVision Update File

11. Click the **Upload & Install** button.


After a moment, the text “Update in progress...” shows on the screen, indicating the update is active. This process may take up to 30 minutes to complete.

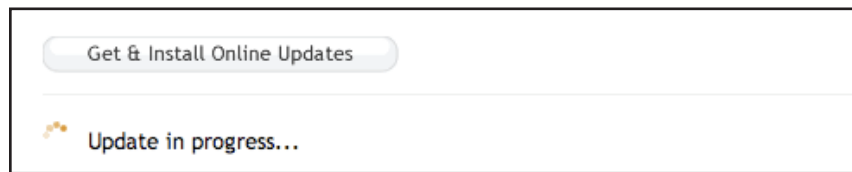

Updating METIVision

IMPORTANT: Do NOT reboot your METIVision or refresh the web browser while the update is installing.

Once the update is finished, the message “Last Update was Successful” is displayed.

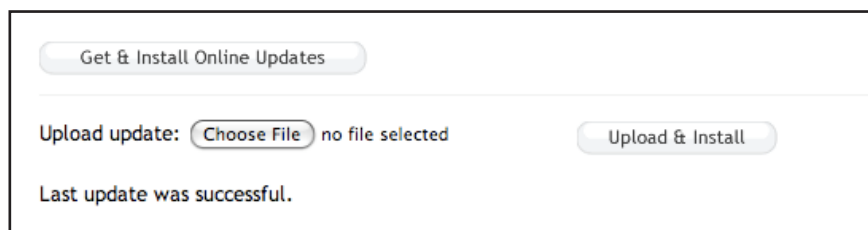

Completing the METIVision Update

12. Exit from the METIVision Main Setup screen by closing the web browser.
13. Remove the Ethernet Cable from the Camera port.


IMPORTANT: If the computer being used for this update is the same as the one being used for the METIVision, the Network settings must be restored. Contact your Network Administrator if you need assistance. Once restored, proceed to Step 14. If using an alternate computer to access METIVision, proceed to Step 14.

14. From the computer used to access METIVision, using the METIVision IP address, access the METIVision Login screen.

The METIVision Login screen appears.

15. To confirm the update is complete, click the **About** box in the lower, right-hand corner of the Login screen.

METIVision Version 1.0.16 is displayed.

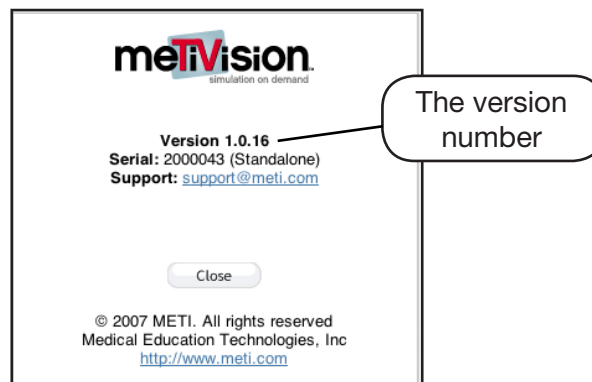

The About Box