


Müse Quick Start Chart


Windows[®] and Macintosh[®] Operating Systems

Once the METIman simulator is connected and powered on, the Müse software can now be launched.

1. With the same computer that is connected to METIman, launch the desired web browser that meets the Müse System Requirements. If using the provided computer, the Müse Start Screen appears automatically when the web browser is launched, and Step 2 can be skipped.
2. If using a computer other than the one supplied, enter the IP Address that appears on the METIman Data Sheet into the web browser's address field. (The METIman Data Sheet is shipped with METIman.)

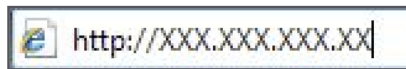


Enter IP Windows

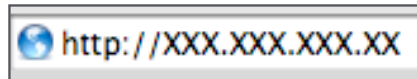

Enter IP Macintosh

The Müse Start Screen appears.

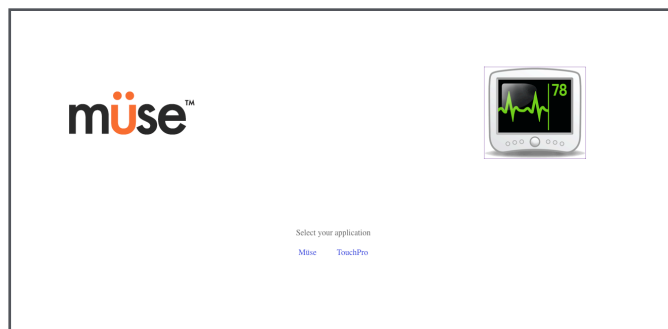

The Müse Start Screen

3. Select **Müse**. The Login Screen appears.



The Login Screen

NOTE: The METIman User Guide is available for download by clicking the **User Guide** link in the bottom, left-hand corner of the screen.

4. Enter the **username** *admin* and the **password** *admin* in the appropriate fields to access the software.



Entering the Username and Password

The software opens to the Home screen by default.